

# TRAVEL SUPPORT

## INSTRUCTIONS



OWL LEASING,™ INC

800-916-4592

OakWorks Leasing  
Travel Support Instructions

Remove QuickLock Face Rest from either the Table Top Support or the Seated Support. Attach black plastic plate with 3-pad face rest cushions.

Slide the Face Rest dowels into the holes located on the Travel Support.

Place Travel Support on your lap making sure the open ends of the U-frame and face rest cushion point toward you. Unlock white cam locks.

Adjust according to comfort and proper face down position. Re-lock cam locks.

NOTE: Sit in the center of the black seat of the vehicle with seatbelt on and travel support in lap.