

OWL MIRROR

INSTRUCTIONS



OWL LEASING, INC.

800-916-4592

OWL Mirror Set-Up Instructions

STEP 1

Remove mirror packing materials. **Be sure to keep the packing materials for return shipment.**

STEP 2

Both large and small mirrors pivot. Adjust the mirrors as shown below.

Line up top of small mirror to be slightly above the black plastic.



Line up large mirror with the back edge of the plastic.

STEP 3

If using with the Seated Support, remove the arm rest pad. Place Mirror on the chair shelf or on table if using the Table Top Support.




STEP 4

Adjust the mirrors.



Use the large mirror to adjust to the height of the object being viewed.

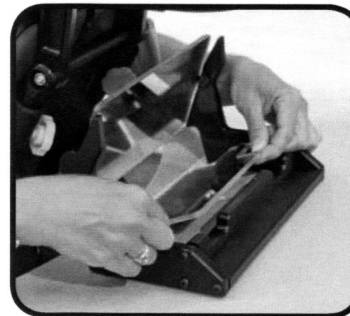

Use the small mirror to adjust to the maximum size and centering of the image.

NOTE: you may need to go back and forth a few times to obtain the best image possible.

OWL MIRROR RE-PACKING INSTRUCTIONS:

Patients are responsible for the safe return of equipment. All items must be packaged properly for the return trip. If there are any questions as to how to repack, call your Personal Care Consultant for assistance. If it is returned damaged, the patient is responsible for replacement costs.